

Enjoy your maternity life! Acupressure bands which relief nausea has landed!



General Medical Equipment

Medical Equipment Notification Number:11B2X10030726201

# サイバンド・ママ PsiBands<sup>®</sup> Drug-Free Wrist Bands for the Relief of Nausea **mama**<sup>™</sup>

**Psi Bands is medical acupressure band which stimulates Nei-kuan point in order to relief unpleasant nausea due to morning sickness and motion sickness.**

[Nei-Kuan] is the deeply involved with the visceral function on the inside of wrist. When people push the point it has effects to relief the nausea from the stomach.

\*About effects there are individual differences.

### Available to adjust pressure and suitable size

Psi Bands mama is able to adjust size with 10 stages like wearing a watch (13cm-19cm of around wrist).

Since the dial is able to adjust stimulating pressure for the point by 4 stages, pregnant woman who has swelling can use it based on her physical condition.

### Able to wash! Easy to care!

Psi Bands mama is waterproof so that you can use it during bath time.

### High Designability

You can choose from 3 designs like you can choose accessory.

FDA FDA (US Food and Drug Administration) Cleared



**Color Play**

JAN:4562384601618  
4 562384 601618

**Crystal Clear**

JAN:4562384601625  
4 562384 601625

**Heart Land**

JAN:4562384601632  
4 562384 601632

## Directions for Use (Both bands should be worn at the same time)



Place one band on the inside of each wrist, 2 and half finger widths above the center of your wrist crease between the wrist tendons. This placement is known as the Nei-kuan acupressure point. The button should be facing in and the dial should be facing out. If the point is not matched to the Nei-kuan point, there is no affection. Please wear the bands just as you would wear a watch.

1. To adjust the band size around the wrist, place the post of the dial through the appropriate band hole. Feed the slack of the band under the slide-able collar, just as you would wear a watch.  
2. To increase or decrease pressure on the Nei-kuan acupressure point, rotate the dial. Please adjust pressure and the best position is that you don't feel hurt.

**[Product name]** Psi Bands mama

**[Medical Equipment Classification]** General Medical Equipment (Class I)  
Medical Equipment Notification Number: 11B2X10030726201

**[Japanese Retail Price]** ¥3,090

**[Material]** TPE,PP \*Psi Bands mama are not made with natural rubber latex

**[Product Size]** W24×D13.5×H235 mm/Weight 7.7g

**[Package Size]** W54×D40×H140 mm/Weight 65.5g

**[Quantity in Inner case]** 3PCS

**[Quantity in Outer case]** 48PCS

**[BOX Size]** Inner Box: W249×D49×H58 mm/Weight 0.22 kg

**[Outer Box]:** W265×D215×H275 mm/Weight 3.76 kg

**[Authorized Representative]** ADVANCE JAPAN

**[Distributor]** TECHXCEL JAPAN CO., LTD.

**[Release Date]** December 15th 2016

**Counter Utensil (Available to Hang Psi Bands mama)**  
Utensil size: W220×D120×H300 (180+120)mm  
Display Quantity: 6PCS (3SKU×2PCS)



Signboard Part  
(Front-Back)

Set of Including  
Utensil

JAN:4562384601700  
4 562384 601700

Only one item production & Product Development.  
**TECHXCEL JAPAN**  
TECHXCEL JAPAN Co.,Ltd



Tokyo Office: 3F Kora Bldg, 1-29-17 Ogawahigashi-cho, Kodaira-shi, Tokyo,187-0031 Japan  
TEL :042-313-2912 FAX:042-313-2913  
http://www.techxcel-japan.com

\*Photos, pictures are sample. Design may change.